



Setting Up & Using LAMP Words for Life

Learn more about [LAMP](https://aacapps.com/home) products at
<https://aacapps.com/home>

LEARNING OBJECTIVES

By the end of the training, you will be able to:

- Identify individuals who would benefit from this app.
- Set-up and personalize vocabulary.
- Maximize use of app for phone communication.

CONSIDERATIONS FOR SELECTION

- Previous experience with Unity language system
- Physical ability to select from 84 location grid
- Consistent layout of the icons
- Vocabulary Builder

LAMP IN ACTION

[○ Click to Watch Video](#)

RATE ENHANCEMENT

CORE WORD VOCABULARY

Vocab												Menu
finished	mine	little	up	yes	good	some	no	down	out	off	bad	
me	my	wear	am	please	that	and	in	what	a	+s	there	
I	we	are	is	were	was	on	to	word	an	the	end	
you	they	new	play	like	work	have	feel	read	more	fast	stop	
it	he	want	all	come	time	do	go	get	big	color	help	
she	look	slow	hear	think	right	said	live	love	follow	ride	put	
CLEAR	not	talk	sit	eat	find	make	need	drink	watch	turn	sleep	

ICON PREDICTION

Back										Menu
← GO BACK		I hope								
		I had	I am							
I		I'm	I've	I'll	I'd					
		I know		I like	I must	I have	I feel	I will		I should
		I want	I did		I was	I do	I'm going		I might	
		I shall		I think	I may			I love		I could
CLEAR	NT					I wish	I need	I can		I would


SELECTING A VOCABULARY FILE

○ [Click to view the video](#)

- 84 One Hit vocabulary
- 84 Transition vocabulary
- 84 Full vocabulary

Let's try out the vocabularies

STORING CUSTOM VOCABULARY

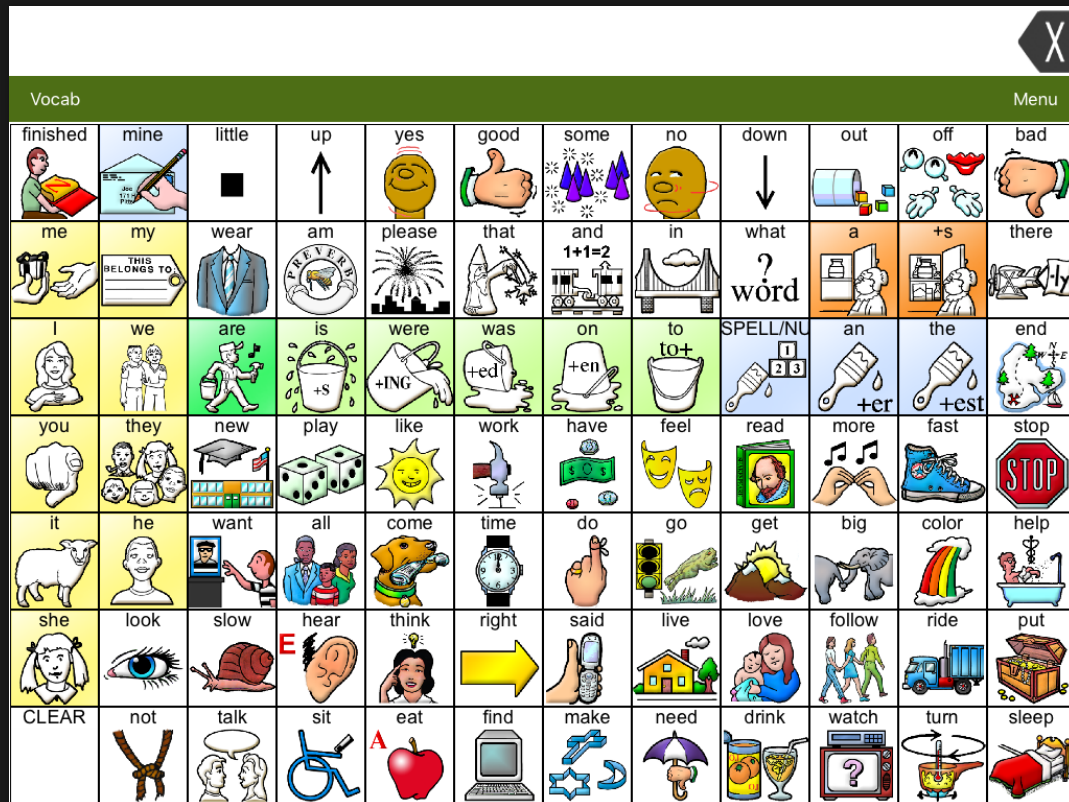

- Go to the **Vocab File** listing by choosing **Vocab**
- Select **Choose New Vocab**
- Select **Menu**
- Select **Copy a Vocab**
- Select your vocabulary file
- Provide a **New Title**
- Select **Save**

WORD FINDER

[Click View Video](#)

- Select **Menu** or the **Spelling** button.
- Select **Word Finder**.
- Type the word you want to find.
- Select the **Find** button.
- The window will identify which keys to press to activate the word.

SOUND SETTINGS: CHANGING THE VOICE

[○ Click to View Video](#)

1. From the app, select **Menu**.
2. Select **Settings**.
3. In the **Speech** section, find **Voice** to select any of the available voices.
4. Select **Done**.

SOUND SETTINGS: TURNING BUTTON CLICKS ON AND OFF

1. From the app, select **Menu**.
2. Select **Settings**.
3. In the **Navigation** section, find **Button Click Sound** and slide it to **On** or **Off**.
4. Select **Done**.

EDITING AN EXISTING BUTTON

○ [Click to View Video](#)

- Go to the page where the button is located.
- Select **Menu**.
- Select **Edit Page**.
- Select the button to be edited.
- Select **Edit This Button**.
- Enter button preferences
- When finished, select **Save**.
- Select **Done** in the top left of the page to exit edit mode.

COMMUNICATION OVER ANY SPEAKERPHONE



IN-PERSON VS. DISTANCE COMMUNICATION



COMMUNICATION WHEN YOU CAN HEAR ME & SEE ME



- Signs
- Facial expressions
- Gestures
- Vocalizations
- Context
- Communication device

COMMUNICATION WHEN YOU CAN ONLY HEAR ME



- Signs
- Facial expressions
- Gestures
- Vocalizations
- Context
- Communication device

PURPOSE OF A PHONE SCRIPT



- Speed up communication
- Get your main point across accurately
- Control the conversation
- Repair breakdowns

CONTENTS OF A PHONE SCRIPT

I need a ride. X

Menu

CLEAR

gs
S
message

Activity Creating a Phone Script Page

- Choose LAMP WFL Full Version


Activity 2 – Making Sentences

- Without using the keyboard feature write this sentence out: I want to go to the library. Please

Activity 3 – Making Sentences


- Without using the keyboard feature write this sentence out: **What time is it?**
- How many buttons did you have to push?

CREATING NEW BUTTONS

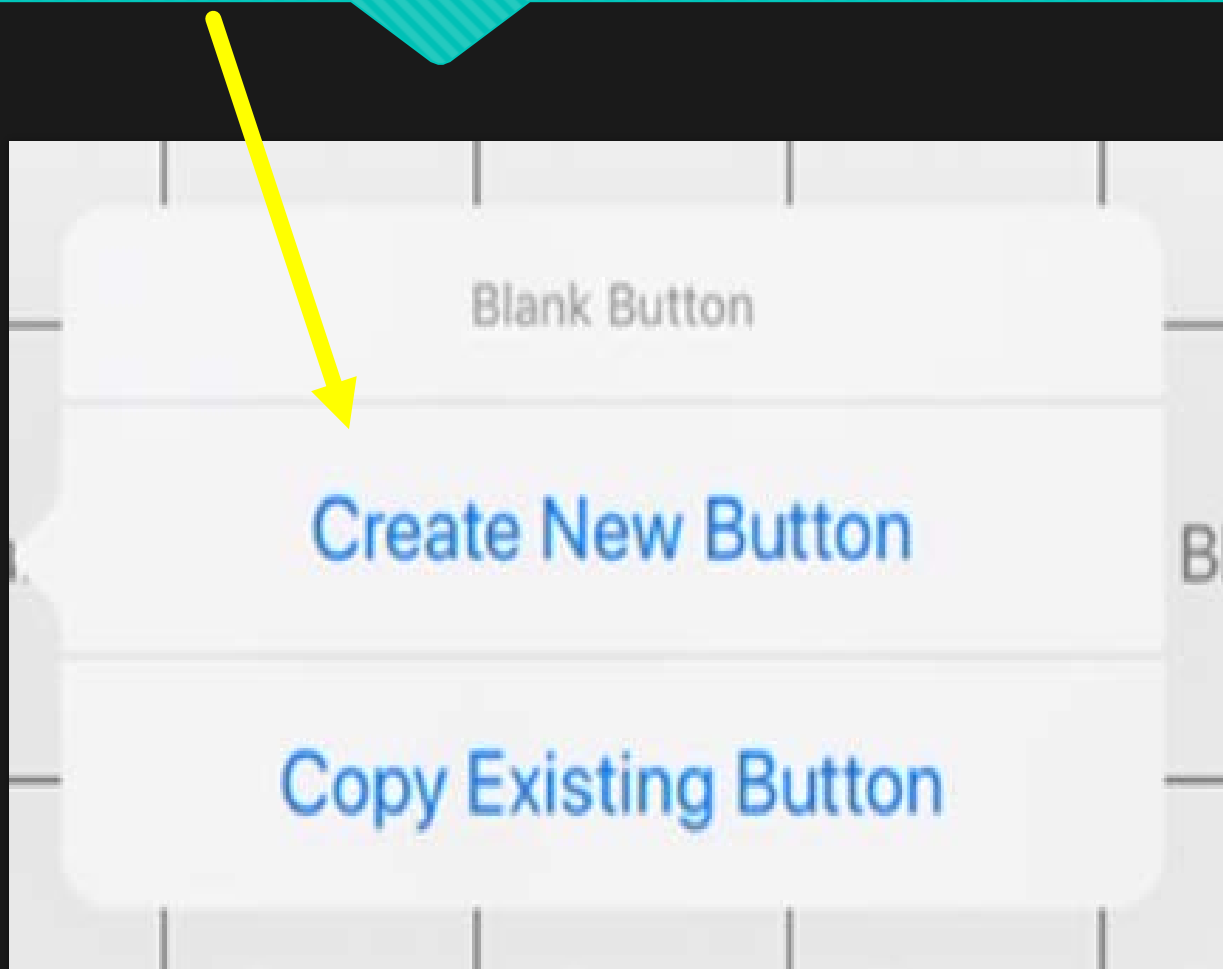

- Select **Menu**.
- Select **Edit Page**.
- Select the location for the new word.
- Select **Create New Button**.


CUSTOMIZE BUTTON PREFERENCES

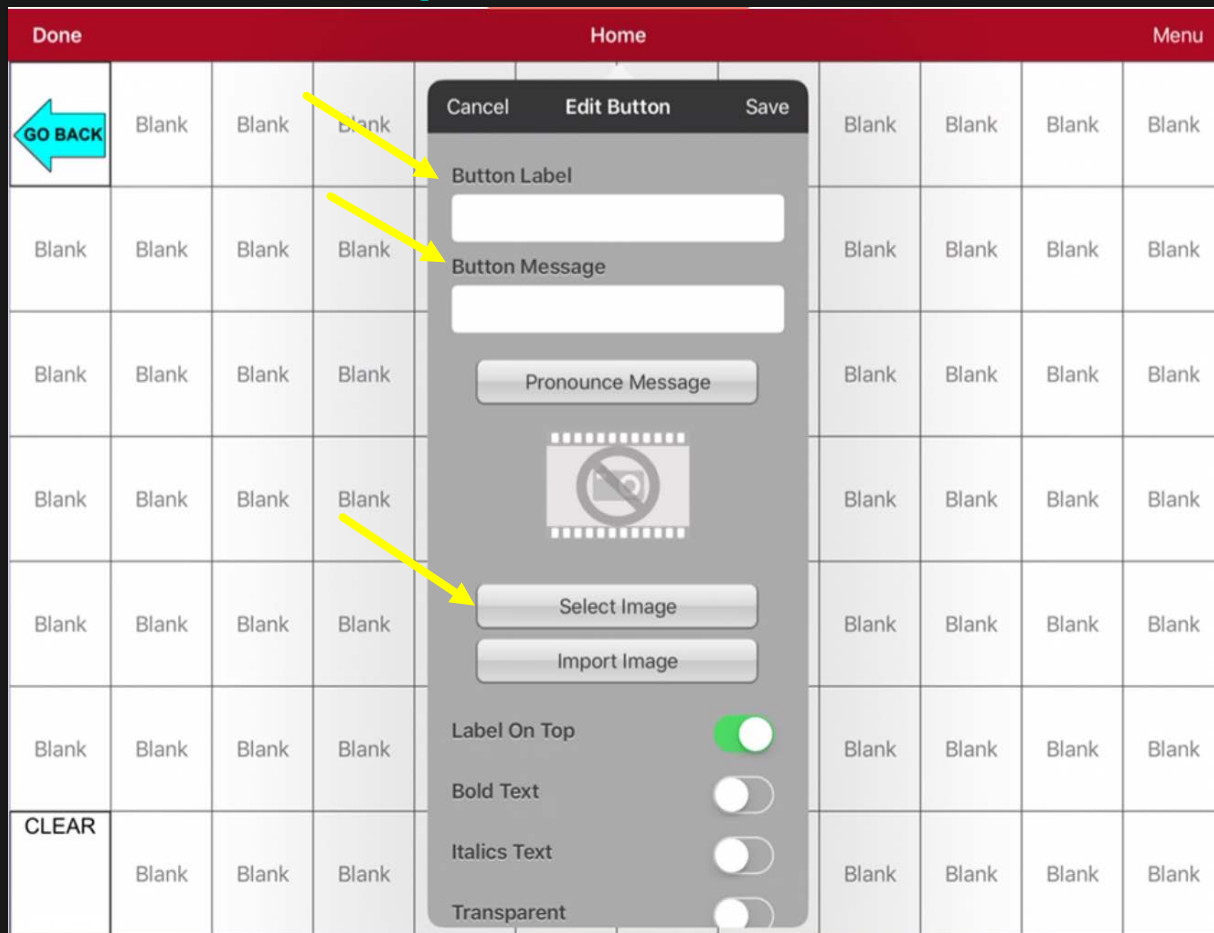

Enter button preferences such as:


- Button Label
- Button Message
- Select or import image
- Font
- Body Color
- Border
- Actions

SAVE CUSTOMIZED BUTTON PREFERENCES

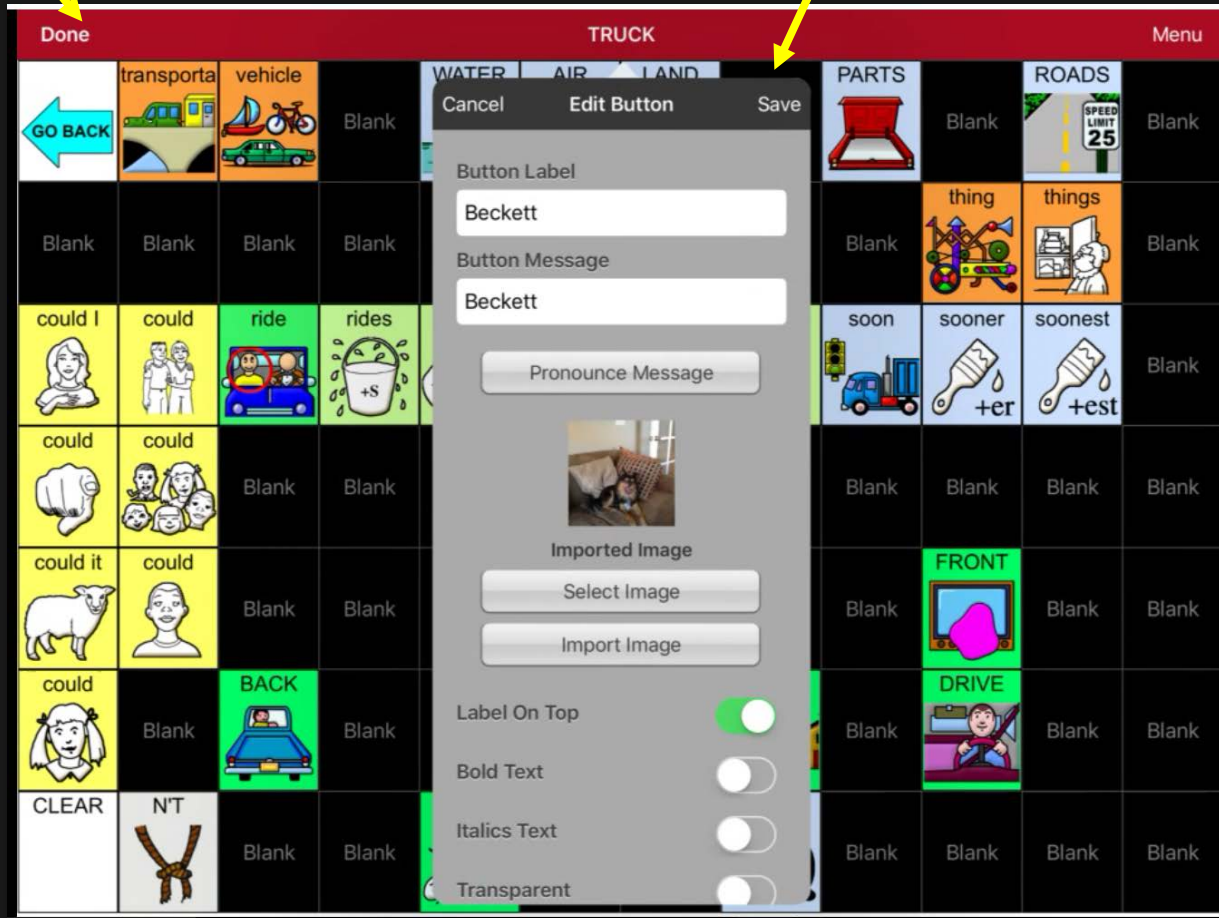

- Select **Save**
- Select **Done**

USING VOCABULARY BUILDER TO TEACH MOTOR PATTERNS

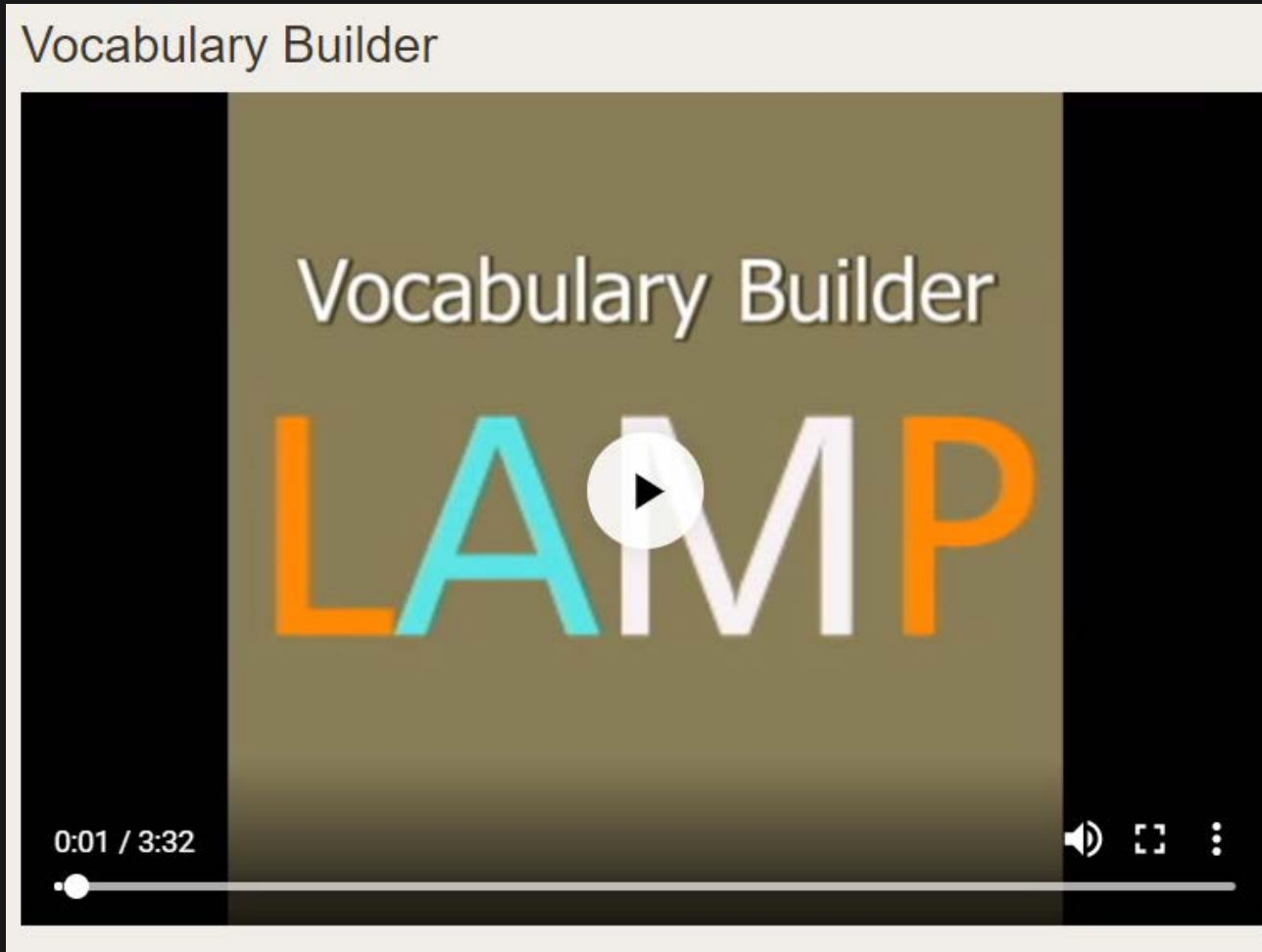

VOCABULARY BUILDER: CREATING A LIST OF WORDS TO TEACH

1. From the app, select **Menu**
2. Select **Vocab Builder**
3. Slide the **On/Off** button at the bottom to **On**.
4. Select **Disable All** on the bottom right of the screen.
5. Either a) **type a word** you want to teach in the search window and then **check the box** next to that word on the list **or** b) select the **Quick Edit** key at the bottom left of the window and type in each word, selecting **Add Word**, after each entry
6. Select **Save**
7. Select **Done**

VOCABULARY BUILDER: SAVING A LIST VIDEO

Saving a Vocabulary List

LAMP


VOCABULARY BUILDER: SAVING A LIST

1. From the app, select **Menu**
2. Select **Vocab Builder**
3. Select **Save List**.
4. Give the list a name in the **Word List File Name** field.
5. Select **Save**
6. Once complete, you will be notified "**File Saved Successfully**".
7. Select **Okay**.
8. Select **Done** to exit the Vocabulary Builder menu.

VOCABULARY BUILDER: LOADING A LIST

1. From the app, select **Menu**
2. Select **Vocab Builder**
3. Select **File List**.
4. Select the list to be loaded
5. Select **Load File**.
6. Select **Done** to exit the Vocabulary Builder menu.

SENDING YOUR MESSAGE: EMAIL & SOCIAL MEDIA

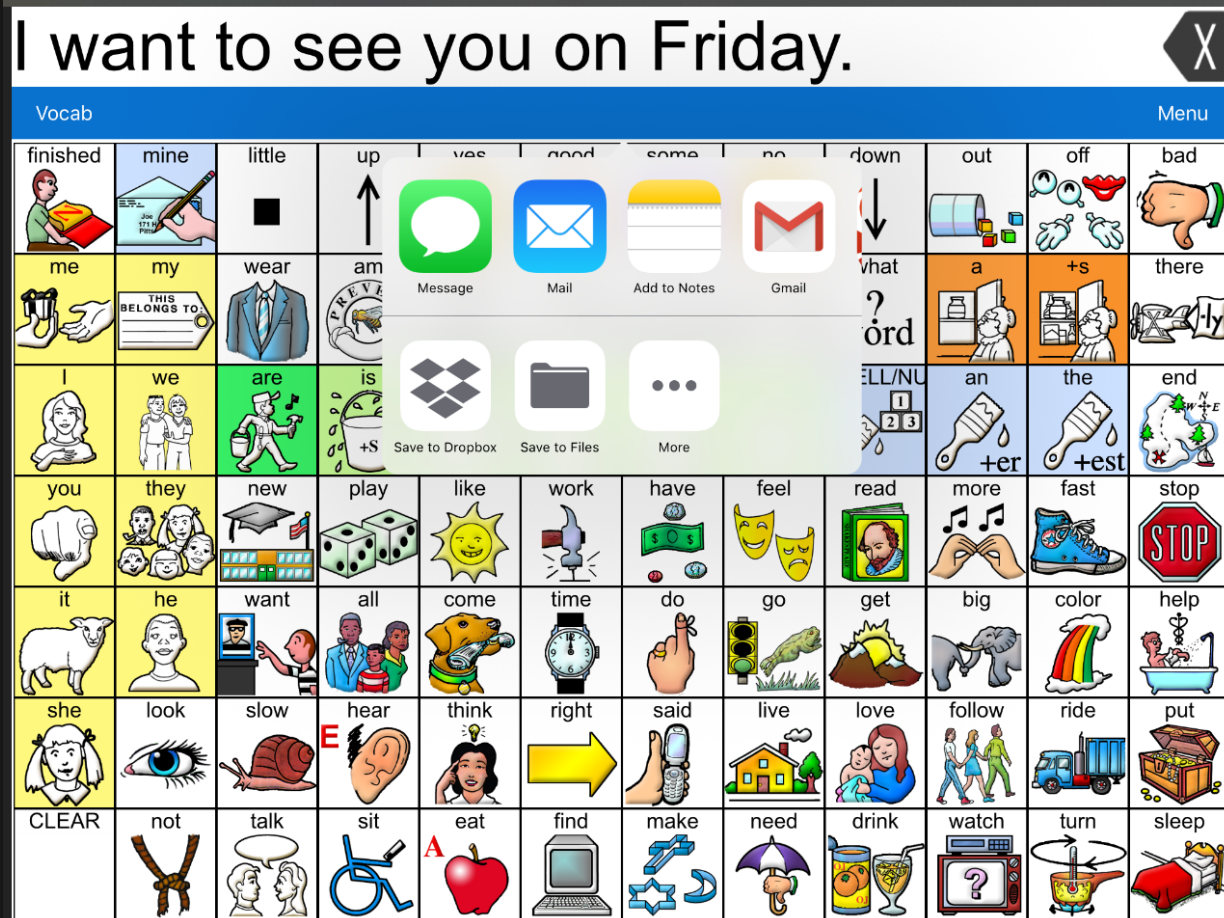

Send your message to via email or social media app by

- Long press on your message.
- Select **Share**
- Select the **app icon** to send your message.

QUESTIONS??

Visit the [LAMP](#) website for the most recent training videos and technical support articles

