



SETTING UP & USING TOUCHCHAT HD WORDPOWER

Learn more about [TouchChat](https://touchchatapp.com/support/touchchat) products at
<https://touchchatapp.com/support/touchchat>

LEARNING OBJECTIVES

By the end of the training, you will be able to:

- Identify individuals who would benefit from this app.
- Set-up and personalize vocabulary.
- Maximize use of app for phone communication.

TOUCHCHAT IN-ACTION

○ Watch Movie

○ Link to TouchChat App Training Videos

CONSIDERATIONS FOR SELECTION

- Symbol supports for literacy needs
- Large range of vocabulary layouts
- Large core word vocabulary
- Customization for language, vision and physical needs
- Spanish vocabulary available

RATE ENHANCEMENT


CORE WORD VOCAB



ICON PREDICTION



WORD PREDICTION

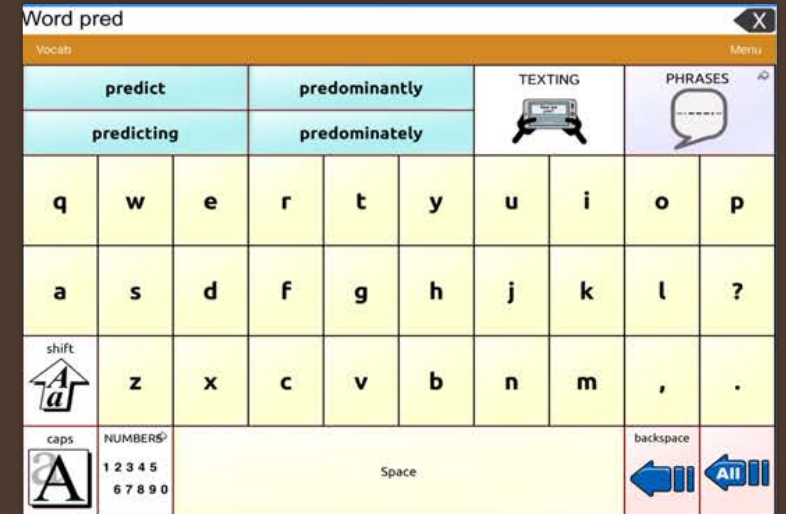

WORD POWER

CORE WORD VOCABULARY

PEOPLE	QUESTN	PLACES	SOCIAL	TIME	GROUPS	DESCRB
I	me	a	the	that	more	ABC 123
my	can	to	come	eat	drink	ACTIONS
it	do	feel	finish	go	help	hurt
you	is	like	listen	need	play	stop
your	don't	.	want	watch	work	with

PEOPLE	QUESTN	PLACES	SOCIAL	TIME	GROUPS	DESCRB
I	me	a	the	that	more	ABC 123
my	can	to	come	eat	drink	ACTIONS
it	do	feel	finish	go	help	hurt
you	is	like	listen	need	play	stop
your	don't	.	want	watch	work	with

PEOPLE	QUESTN	ACTIONS	PLACES	SOCIAL	TIME	GROUPS	DESCRB	good	
ABC 123	I	me	don't	to	be-	come	any-	a	more
clear	my	can	eat	drink	finish	go	this	the	that
.	it	do	get	help	know	listen	about	and	at
delete wd	it	do	get	help	know	listen	about	and	at
-s	you	is	like	need	play	stop	for	in	with
we	your	have	think	want	watch	work	of	on	out

clear	-s								
q	w	e	r	t	y	u	i	o	p
.	a	s	d	f	g	h	j	k	l
delete	Shift 123	z	x	c	v	b	n	m	bksp
I	me	to	PEOPLE	QUESTN	Space	PLACES	SOCIAL	TIME	GROUPS
my	is	be-	come	go	the	a-	and	at	DESCRB
it	can	do	drink	eat	that	for	in	of	more
you	your	ACTIONS	play	want	this	on	out	with	any- or every- if

PEOPLE	QUESTN	PLACES	TIME	SOCIAL	DESCRB	good	more	-ed	-ing	-s	GROUPS
I	me	Q	W	E	R	T	Y	U	I	O	P
my	don't	A	S	D	F	G	H	J	K	L	bksp
it	not	Shift 123	Z	X	C	space	V	B	N	M	.
he	is	to	be-	come	eat	drink	feel	at	and	a-	the
she	are	do	did	go	get	help	like	for	in	on	that
they	was	can	could	make	need	play	put	read	of	off	this
we	have	will	would	say	see	stop	take	talk	up	out	any- or every- if
you	your	ACTIONS	want	tell	think	turn	watch	work	down	with	clear

DO YOU LIKE THICK CRUST?

PEOPLE	QUESTN	SOCIAL	PLACES	-ed	-ing	-s	TIME	GROUPS
I	me	my	don't	not	good	more	okay	really
it	are	is	to	be	ask	buy	come	yes
he	can	can't	drink	eat	feel	finish	go	that
she	could	could't	get	help	hurt	know	like	about
they	do	did	listen	look	love	make	need	because
we	have	had	play	put	read	say	see	in
your	was	were	stop	take	talk	tell	think	on
you	will	would	ACTIONS	turn	want	watch	work	off

SELECTING A VOCABULARY FILE



1. Tap **Vocab** in upper left hand corner
2. Press **Choose new vocab**
3. Scroll to find your vocabulary file.

Let's try out the vocabularies

STORING CUSTOM VOCABULARY

In order to edit an original vocabulary file, a copy must be made first.

1. Go to the Vocab File listing by choosing **Vocab** and **Choose New Vocab**
2. Choose **Menu**
3. Choose **Copy a Vocab**
4. Choose the **Vocab File** to copy
5. Give it a **Name** and **New Description**
6. Choose **Save**

SOUND SETTINGS: CHANGING THE VOICE

1. From the app, Select **Menu**
2. Select **Settings**
3. In the **Speech** section, find **Voice** to select any of the available voices
4. Select **Done**

SOUND SETTINGS: TURNING BUTTON CLICKS ON AND OFF

1. From the app, Select **Menu**
2. Select **Settings**
3. In the **Navigation** section, find **Button Click Sound**.
4. To turn the button click off, slide the button to the left.
5. To turn the button click on, slide the button to the right.
6. Select **Done**

CHANGING YOUR TOUCH SETTINGS: ACTIVATE ON RELEASE

1. Tap **Menu**
2. Select **Settings**
3. Scroll until you find the **Buttons** section
4. Slide the button to the left to turn it off.
5. Slide the button to the right to turn it on.
6. Select **Done**

COMMUNICATION OVER ANY SPEAKERPHONE

[Click to View Movie](#)



IN-PERSON VS. DISTANCE COMMUNICATION



COMMUNICATION WHEN YOU CAN HEAR ME & SEE ME




- Signs
- Facial expressions
- Gestures
- Vocalizations
- Context
- Communication device

COMMUNICATION WHEN YOU CAN ONLY HEAR ME




- Signs
- Facial expressions
- Gestures
- Vocalizations
- Context
- Communication device

PURPOSE OF A PHONE SCRIPT

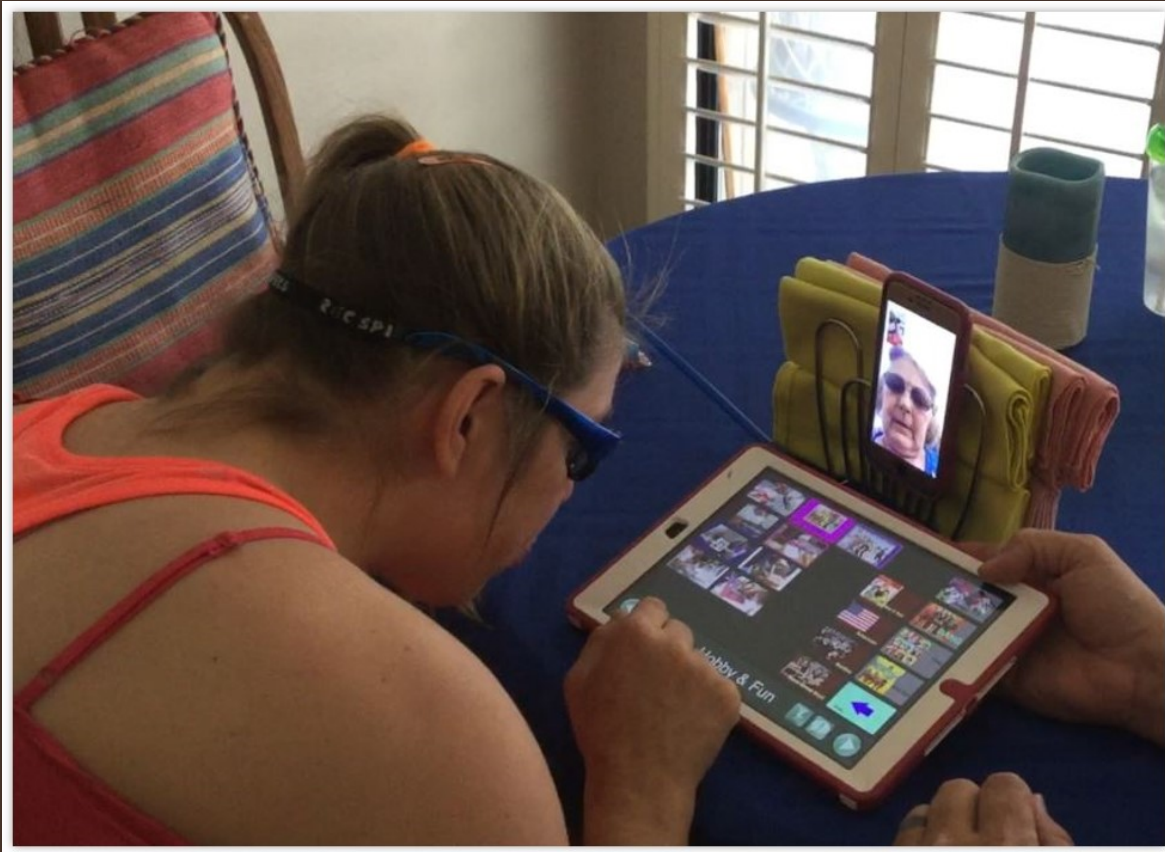

- Speed up communication
- Get your main point across accurately
- Control the conversation
- Repair breakdowns

CONTENT OF A PHONE SCRIPT

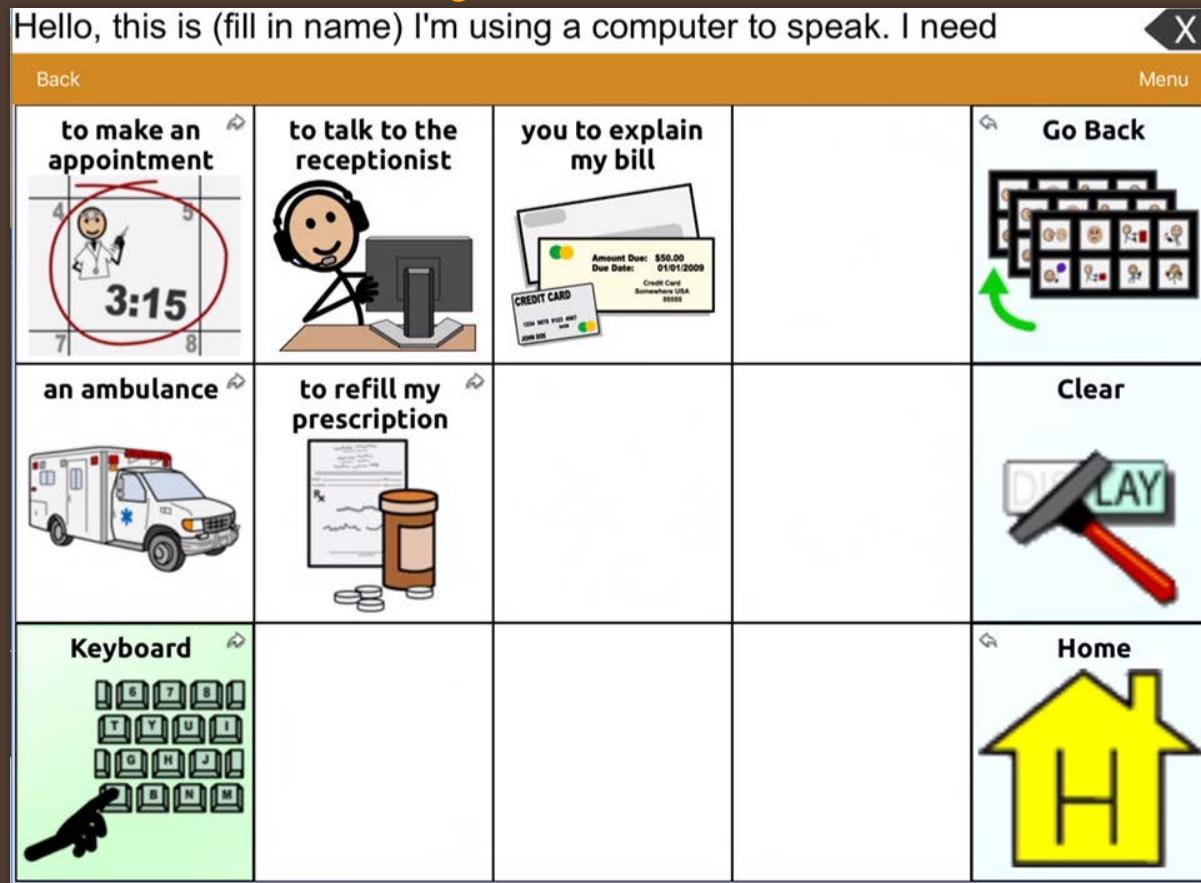

- Manage the call
- Greetings & Closings
- Repair breakdowns
- Share the main message

EDITING A BUTTON

○ Click to View Movie

1. Go to the page where the button is located.
2. Select **Menu**.
3. Select **Edit Page**.
4. Select the button to be edited.
5. Select **Edit This Button**.

CUSTOMIZE BUTTON PREFERENCES



Enter button preferences such as:

- Button Label
- Button Message
- Select or import image
- Font
- Body Color
- Border
- Actions

SAVE CUSTOMIZED BUTTON PREFERENCES



- Select **Save**
- Select **Done**

BUTTON ACTIONS: NAVIGATE TO A DIFFERENT PAGE

1. Select **Menu**
2. Select **Edit Page**
3. Tap a button to edit
4. Select **Edit This Button**
5. Scroll down to **Button Actions**
6. Select **Navigate**
7. Choose Your Page
8. Select animation option
9. Select **Save and** Select **Done**

COPY AND PASTE A BUTTON



1. Tap the **Menu** button and then **Edit Page**.
2. Touch the button you would like to copy.
3. Select **Copy Button**. Tap **Done**.
4. Go to the page where you want to "paste" the copied button.
5. Once there, tap the **Menu** button in the top right of the screen and choose **Edit Page**.
6. Tap on the location you want to paste to.
7. Select **Paste Button** from the menu.

Lets Practice # 1 Word Power 42

- Write this sentence: "I want to eat pizza with onions pepperoni sausage and extra mushrooms.
- DO NOT USE KEYBOARD!

Lets Practice # 2 – Word power 42

○ My Family go for a walk to the park tonight



Lets Practice # 3 Word Power 140

- Write this sentence: "I wont eat tomatoes
- DO NOT USE KEYBOARD!

Lets Practice # 4 Word Power 140

- Write this sentence: "I like to visit the river and camp"
- DO NOT USE KEYBOARD!

SENDING YOUR MESSAGE EMAIL & SOCIAL MEDIA

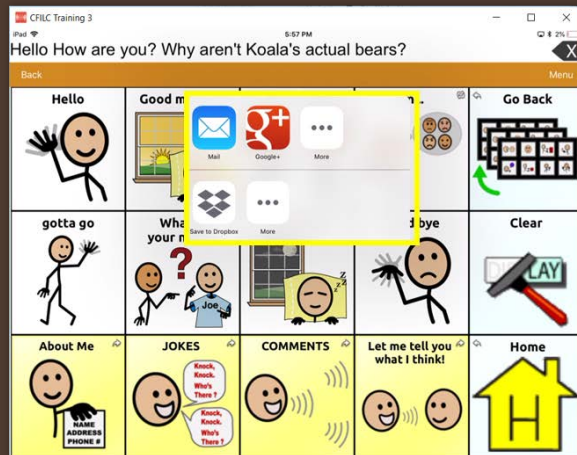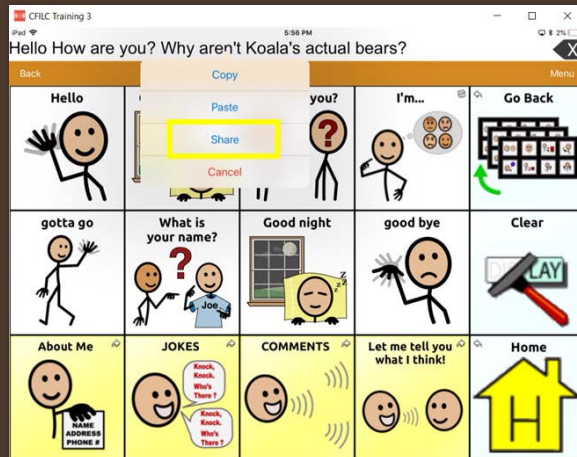


1. Select **Menu**
2. Select **Settings**
3. Scroll down and turn on **Allow Social Networking**
4. Select **Done**
5. Type your message
6. **Touch and hold** the speech display bar
7. Select **Share**
8. Choose the service you want to use
9. Select **Done**

Bilingual Options

There are two ways of using TouchChat HD with languages.

1. The first option is make the voice speak a different language than the buttons by opening settings and changing the voice.
2. The second option is to open the list of vocabulary files, and scroll down to Spanish. Tap Spanish and it should open a listing of vocabulary files that can be used only in Spanish or in Spanish and English.

BACKING UP YOUR EDITED VOCABULARY with DropBox

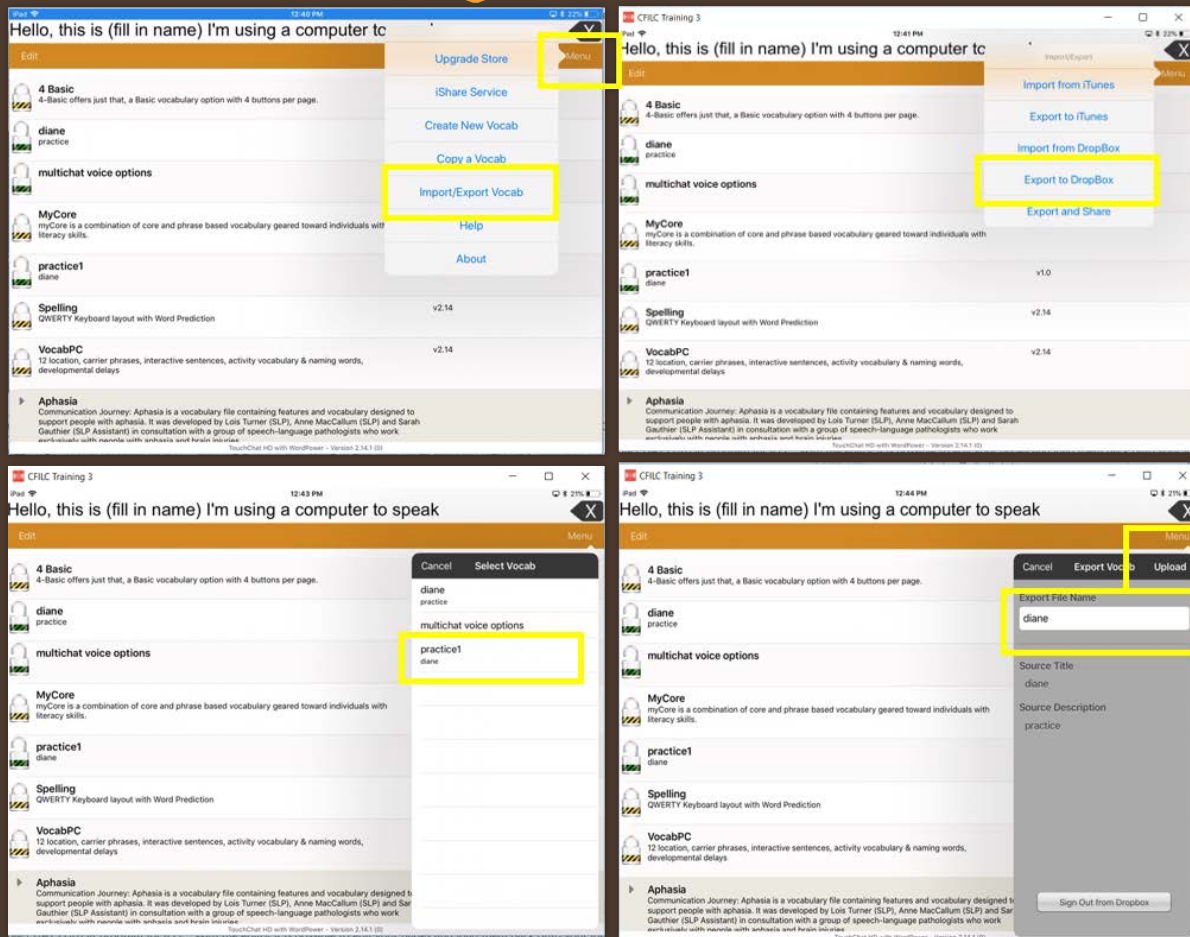

1. From the choose vocab screen, select **Menu**.
2. Select **Import/Export Vocab**.
3. Tap **Export to DropBox**
4. Select the vocab list to back up.
5. Change the file name or add a date if needed.
6. Select **Upload**.

QUESTIONS??

Visit the [TouchChat](https://touchchatapp.com/support/touchchat) website for the most recent training videos and technical support articles

<https://touchchatapp.com/support/touchchat>



CTEC

Communication Technology
Education Center

www.ctecAAC.org